INSTANT DRY MOUTH RELIEF

www.bioxtra.info

INSTANT DRY MOUTH RELIEF

bioXtra
INFORMATION & WARNINGS BEFORE USING
BIOXTRA DRY MOUTH ULTRA MILD MOUTH RINSE

en

bioXtra Dry Mouth Ultra Mild Mouthrinse (30 ml)

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In case of serious incidents in relation to the use of this product, report immediately to the manufacturer and your local competent authority.**Incompatibilities / Interactions with other medication, food or alcohol:**

None known.

bioXtra Dry Mouth Ultra Mild Mouthrinse (30 ml)

Please read the following instructions for use carefully, they contain important information.**HOW TO USE BIOXTRA DRY MOUTH ULTRA MILD MOUTH RINSE**
May be used as and when symptoms occur. If you feel an improvement, continue using the product for a few more days before slowly reducing the number of applications. If symptoms return, use for as long as necessary paying attention to the recommended daily dose.

bioXtra Dry Mouth Ultra Mild Mouthrinse should only be used in accordance with these instructions or with those given by your pharmacist / health care professional.

If you experience any undesirable effect when using the product, seek medical advice.

Intended Users:
Adults and Children 12 years and over.
For children under 12 years of age, parent or guardian should seek medical advice before use.**Not suitable for infants. Keep out of reach of children.****Recommended daily dose: 50ml**
10 ml up to maximum of 5 times per 24h period (measure each dose using 10 ml line on measuring cup supplied).

Do not use this product if the bottle or its seal are broken.

WHAT IS BIOXTRA DRY MOUTH ULTRA MILD MOUTH RINSE, AND WHEN SHOULD IT BE USED?

bioXtra Dry Mouth Ultra Mild Mouthrinse is a medical device. The intended purpose is to alleviate the symptoms and effects of dry mouth (xerostomia) whether due to medical conditions, medical treatments or medical interventions.

What is Dry Mouth (xerostomia) and what are the benefits of using bioXtra Dry Mouth Ultra Mild Mouthrinse?

Natural saliva contains a complex balance of essential components, which protect the sensitive tissues of your teeth and gums.

A reduction or halt in saliva flow can lead to a condition known as dry mouth, clinically known as xerostomia. The effects of dry mouth may include discomfort, poor oral health, bad breath, difficulty with speech and swallowing and reduction in quality of life.

bioXtra Dry Mouth Ultra Mild Mouthrinse has been formulated to alleviate dryness in the oral cavity by supplementing the natural moisturising and lubricating systems in saliva with an instant soothing effect. bioXtra Dry Mouth Ultra Mild Mouthrinse also contains enzymes similar to those found in natural saliva to help maintain a healthy microenvironment in the mouth.

bioXtra Dry Mouth Ultra Mild Mouthrinse does not contain alcohol or detergents.

When to use bioXtra Dry Mouth Ultra Mild Mouthrinse (indications):

There are multiple causes of xerostomia. These include conditions such as:

- Sjögren's Syndrome
- Systemic health conditions, such as high blood pressure, diabetes, and auto-immune diseases
- Cancer therapy
- Hormonal imbalance
- Stress and anxiety
- Use of multiple medication

bioXtra Dry Mouth Ultra Mild Mouthrinse can be used to alleviate the symptoms of xerostomia originating from all known causes.

IMPORTANT INFORMATION & WARNINGS BEFORE USING BIOXTRA DRY MOUTH ULTRA MILD MOUTH RINSE

Contains Sodium Monofluorophosphate (1500 ppm as F). In case of intake of fluoride from other sources, consult a dentist or doctor.

When not to use bioXtra Dry Mouth Ultra Mild Mouthrinse (contra-indications):

Do not use bioXtra Dry Mouth Ultra Mild Mouthrinse if you are allergic to any of the ingredients. Contains Lactose, not suitable for patients with lactose intolerance.

Risk of allergy:

Contains traces of milk and egg white proteins, known to cause allergy in rare cases.

Contains traces of menthol, present in the aroma, do not use if you are allergic to mint or menthol.

In case of a suspected allergic reaction seek immediate medical assistance.

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Please read the following instructions for use carefully, they contain important information.**HOW TO USE BIOXTRA DRY MOUTH ULTRA MILD MOUTH RINSE**
May be used as and when symptoms occur. If you feel an improvement, continue using the product for a few more days before slowly reducing the number of applications. If symptoms return, use for as long as necessary paying attention to the recommended daily dose.

bioXtra Bain de Bouche Ultra Mild Mouthrinse should only be used in accordance with these instructions or with those given by your pharmacist / health care professional.

If you experience any undesirable effect when using the product, seek medical advice.

Intended Users:
Adults and Children 12 years and over.
For children under 12 years of age, parent or guardian should seek medical advice before use.**Not suitable for infants. Keep out of reach of children.****Recommended daily dose: 50ml**
10 ml up to maximum of 5 times per 24h period (measure each dose using 10 ml line on measuring cup supplied).

Do not use this product if the bottle or its seal are broken.

WHAT IS BIOXTRA DRY MOUTH ULTRA MILD MOUTH RINSE, AND WHEN SHOULD IT BE USED?

bioXtra Bain de Bouche Ultra Mild Mouthrinse is a medical device. The intended purpose is to alleviate the symptoms and effects of dry mouth (xerostomia) whether due to medical conditions, medical treatments or medical interventions.

What is Dry Mouth (xerostomia) and what are the benefits of using bioXtra Dry Mouth Ultra Mild Mouthrinse?

Natural saliva contains a complex balance of essential components, which protect the sensitive tissues of your teeth and gums.

A reduction or halt in saliva flow can lead to a condition known as dry mouth, clinically known as xerostomia. The effects of dry mouth may include discomfort, poor oral health, bad breath, difficulty with speech and swallowing and reduction in quality of life.

bioXtra Bain de Bouche Ultra Mild Mouthrinse has been formulated to alleviate dryness in the oral cavity by supplementing the natural moisturising and lubricating systems in saliva with an instant soothing effect. bioXtra Dry Mouth Ultra Mild Mouthrinse also contains enzymes similar to those found in natural saliva to help maintain a healthy microenvironment in the mouth.

bioXtra Bain de Bouche Ultra Mild Mouthrinse does not contain alcohol or detergents.

When to use bioXtra Dry Mouth Ultra Mild Mouthrinse (indications):

There are multiple causes of xerostomia. These include conditions such as:

- Sjögren's Syndrome
- Systemic health conditions, such as high blood pressure, diabetes, and auto-immune diseases
- Cancer therapy
- Hormonal imbalance
- Stress and anxiety
- Use of multiple medication

bioXtra Bain de Bouche Ultra Mild Mouthrinse can be used to alleviate the symptoms of xerostomia originating from all known causes.

IMPORTANT INFORMATION & WARNINGS BEFORE USING BIOXTRA DRY MOUTH ULTRA MILD MOUTH RINSE

Contains Sodium Monofluorophosphate (1500 ppm as F). In case of intake of fluoride from other sources, consult a dentist or doctor.

When not to use bioXtra Dry Mouth Ultra Mild Mouthrinse (contra-indications):

Do not use bioXtra Dry Mouth Ultra Mild Mouthrinse if you are allergic to any of the ingredients. Contains Lactose, not suitable for patients with lactose intolerance.

Risk of allergy:

Contains traces of milk and egg white proteins, known to cause allergy in rare cases.

Contains traces of menthol, present in the aroma, do not use if you are allergic to mint or menthol.

In case of a suspected allergic reaction seek immediate medical assistance.

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