bioXtra products help ease the symptoms and effects of dry mouth (xerostomia). They contain natural enzymes, moisturisers, fluoride and xylitol (see individual product labels for details).

**bioXtra products are for oral use only.**

**bioXtra products contain traces of milk and egg-white proteins. Do not use if allergic to any of the ingredients.**

*Natural saliva contains a complex balance of essential components, which protect the sensitive tissues of your teeth and gums.*

Any reduction in saliva flow can lead to a condition known as dry mouth, or clinically called xerostomia. Dry mouth can disturb your mouth's natural balance and may contribute to discomfort, poor oral health, bad breath and even affect your general health and well-being.

bioXtra is a range of oral care products which have been specially formulated to help relieve dryness in the oral cavity and regulate the microenvironment of the mouth. They supplement the natural protective and lubricating systems in saliva which are essential for a healthy, comfortable mouth.

And because dry mouth (xerostomia) can cause sensitive, sore gums, for added comfort bioXtra products contain no alcohol, menthol, strong flavours or detergents.

bioXtra products comfort and protect dry, sore or sensitive mouths!

### 3 SIMPLE STEPS TO A MORE COMFORTABLE MOUTH

Special ingredients in bioXtra provide an immediate feeling of moisture and help regulate the microenvironment of the mouth.

**STEP 1 – CLEANSING**  
At least twice daily brush with **bioXtra Dry Mouth Mild Toothpaste**.

**STEP 2 – RINSING**  
Rinse with **bioXtra Dry Mouth Ultra Mild Mouthrinse** after brushing and, if desired, at regular intervals during the day for extra freshness and comfort.

**STEP 3 – MOISTURISING**  
For long-lasting lubrication and comfort, especially at night, apply **bioXtra Dry Mouth Oral Gel** to gums or under dentures.

For convenience and immediate moisturisation during the day spray **bioXtra Dry Mouth Gel Mouthspray** directly into the mouth as often as required.

Used together daily, bioXtra's unique formulations gently ease the symptoms of dry mouth (xerostomia), leave your mouth feeling fresh and comfortable, day and night, and help promote oral health.

#### bioXtra Dry Mouth Mild Toothpaste

bioXtra Dry Mouth Mild Toothpaste gently supplements natural saliva by regulating the microenvironment of the mouth. The mild tasting formulation guards against plaque build-up, strengthens enamel, helps prevent the formation of dental caries and ease the symptoms and effects of dry mouth (xerostomia). It helps maintain a healthy mouth and promote fresh breath.

bioXtra Dry Mouth Mild Toothpaste has a low foaming effect.

**Directions for use:** Brush teeth and gums with bioXtra Dry Mouth Mild Toothpaste at least twice daily using a soft toothbrush. Close cap after use.

Contains sodium monofluorophosphate (max. 1500 ppm F). Children of 6 years and younger: use a pea-sized amount for supervised brushing to minimize swallowing. In case of intake of fluoride from other sources consult a dentist or doctor.

#### bioXtra Dry Mouth Ultra Mild Mouthrinse

bioXtra Dry Mouth Ultra Mild Mouthrinse gently supplements natural saliva by regulating the microenvironment of the mouth. The mild tasting formulation strengthens enamel, helps fight against the formation of dental caries and ease the symptoms and effects of dry mouth (xerostomia). It helps maintain your mouth's natural balance and cleanse, refresh and lubricate your mouth.

The bioXtra Dry Mouth Ultra Mild Mouthrinse is alcohol and menthol free.

**Directions for use:** Use measuring cup. After brushing teeth, use approx. 10ml, rinse for 30 seconds and spit out. No need to rinse with water. Use regularly throughout the day to maintain comfort and freshness. Do not swallow. Do not exceed five times per day. Close cap after use.

Contains sodium monofluorophosphate (max 1500 ppm as F). Not suitable for children of 6 years and under. Supervise children as necessary until capable of using without supervision. Keep out of the reach of children.

#### bioXtra Dry Mouth Oral Gel

bioXtra Dry Mouth Oral Gel acts as an effective substitute for natural saliva by regulating the microenvironment of the mouth. The unique formulation contains intensive moisturisers which soothe and lubricate for hours at a time and ease the symptoms and effects of dry mouth (xerostomia). It helps promote healthy teeth and comfortable gums.

**Directions for use:** Wash hands thoroughly before use. Apply gel with your

finger or a clean swab to gums and tongue or under dentures whenever mouth feels dry, especially at night. Remove or spit out the excess. Repeat as necessary. Close cap after use.

#### bioXtra Dry Mouth Gel Mouthspray

bioXtra Dry Mouth Gel Mouthspray gently supplements natural saliva by regulating the microenvironment of the mouth. The formulation quickly and effectively moistens, soothes and eases oral discomfort of dry mouth (xerostomia). bioXtra Dry Mouth Gel Mouthspray helps strengthen enamel and fight against the formation of dental caries, whilst promoting fresh breath and comfortable gums.

**Directions for use:** Spray onto gums, tongue and lips as often as required. Especially practical and convenient for daytime use. In case of direct contact of nozzle with mouth surfaces, wash after use. Remove or spit out the excess. Close cap after use.

Contains Sodium Monofluorophosphate (max. 150 ppm F). For children under 7 years of age, supervise use, minimise swallowing and in case of intake of fluoride from other sources, consult a dentist or doctor.

**Useful information for diabetics in case of accidental swallowing.**  
Although bioXtra Dry Mouth products are not intended to be swallowed, the following information may be useful for diabetics in case of accidental ingestion.

Only the following bioXtra products contain Dextrose:

- bioXtra Dry Mouth Oral Gel contains 0,0028g Dextrose per gram of Gel.
- bioXtra Dry Mouth Mild Toothpaste contains 0,0025g Dextrose per gram of Toothpaste.

In case of serious incidents, report to the manufacturer and competent authority.

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