bioXtra products help ease the symptoms and effects of dry mouth (xerostomia). They contain natural enzymes, moisturisers, fluoride and xylitol (see individual product labels for details).

bioXtra products are for oral use only.

bioXtra products contain traces of milk and egg-white proteins. Do not use if allergic to any of the ingredients.

Natural saliva contains a complex balance of essential components, which protect the sensitive tissues of your teeth and gums.

Any reduction in saliva flow can lead to a condition known as dry mouth, or clinically called xerostomia. Dry mouth can disturb your body's natural balance and may contribute to discomfort, poor oral health, bad breath and even affect your general health and well-being.

bioXtra is a range of oral care products which have been specially formulated to help relieve dryness in the oral cavity and regulate the microenvironment of the mouth. They supplement the natural protective and lubricating systems in saliva which are essential for a healthy, comfortable mouth.

And because dry mouth (xerostomia) can cause sensitive, sore gums, for added comfort bioXtra products contain no alcohol, menthol, strong flavours or detergent.

bioXtra products comfort and protect dry, sore or sensitive mouths!

3 SIMPLE STEPS TO A MORE COMFORTABLE MOUTH

Special ingredients in bioXtra provide an immediate feeling of moisture and help regulate the microenvironment of the mouth.

STEP 1 – CLEANSING
At least twice daily brush with bioXtra Dry Mouth Mild Toothpaste.

STEP 2 – RINSING
Rinse with bioXtra Dry Mouth Ultra Mild Mouthrinse after brushing and, if desired, at regular intervals during the day for extra freshness and comfort.

STEP 3 – MOISTURISING
For long-lasting lubrication and comfort, especially at night, apply bioXtra Dry Mouth Oral Gel to gums or under dentures.

For convenience and immediate moisturisation during the day spray bioXtra Dry Mouth Gel Mouthspray directly into the mouth as often as required.

Used together daily, bioXtra's unique formulations gently ease the symptoms of dry mouth (xerostomia), leave your mouth feeling fresh and comfortable, day and night, and help promote oral health.

bioXtra Dry Mouth Mild Toothpaste
bioXtra Dry Mouth Mild Toothpaste gently supplements natural saliva by regulating the microenvironment of the mouth. The mild tasting formulation guards against plaque build-up, strengthens enamel, helps prevent the formation of dental caries and ease the symptoms and effects of dry mouth (xerostomia). It helps maintain a healthy mouth and promote fresh breath.

bioXtra Dry Mouth Mild Toothpaste has a low foaming effect.

Directions for use: Brush teeth and gums at least 2 times daily using a soft toothbrush. Close cap after use.

Contains sodium monofluorophosphate (max. 1500 ppm as F⁻). Children of 6 years and younger: use a pea-sized amount for supervised brushing to minimise swallowing. In case of intake of fluoride from other sources consult a dentist or doctor.

bioXtra Dry Mouth Ultra Mild Mouthrinse
bioXtra Dry Mouth Ultra Mild Mouthrinse gently supplements natural saliva by regulating the microenvironment of the mouth. The unique formulation contains intensive moisturisers which soothe and lubricate for hours at a time and ease the symptoms and effects of dry mouth (xerostomia). It helps maintain your mouth's natural balance and cleanse, refresh and lubricate your mouth.

The bioXtra Dry Mouth Ultra Mild Mouthrinse is alcohol and menthol free.

Directions for use: Use measuring cup. After brushing teeth, use approx. 10ml, rinse for 30 seconds and spit out. No need to rinse with water. Use regularly throughout the day to maintain comfort and freshness. Do not swallow. Do not exceed five times per day. Close cap after use.

Contains sodium monofluorophosphate (max 1500 ppm as F⁻). Not suitable for children of 6 years and under. Supervise children as necessary until capable of using without supervision. Keep out of the reach of children.

bioXtra Dry Mouth Oral Gel
bioXtra Dry Mouth Oral Gel acts as an effective substitute for natural saliva by regulating the microenvironment of the mouth. The formulation contains intensive moisturisers which soothe and lubricate for hours at a time and ease the symptoms and effects of dry mouth (xerostomia). It helps promote healthy teeth and comfortable gums.

Directions for use: Wash hands thoroughly before use. Apply gel with your finger or a clean swab to gums and tongue or under dentures whenever mouth feels dry, especially at night. Remove or spit out the excess. Repeat as necessary. Close cap after use.

bioXtra Dry Mouth Gel Mouthspray
bioXtra Dry Mouth Gel Mouthspray gently supplements natural saliva by regulating the microenvironment of the mouth. The formulation quickly and effectively moisturises, soothes and eases oral discomfort of dry mouth (xerostomia). bioXtra Dry Mouth Gel Mouthspray helps strengthen enamel and fight against the formation of dental caries, whilst promoting fresh breath and comfortable gums.

Directions for use: Spray onto gums, tongue and lips as often as required. Especially practical and convenient for daytime use. In case of direct contact of nozzle with mouth surfaces, wash after use. Remove or spit out the excess. Close cap after use.

Contains Sodium Monofluorophosphate (max. 1500 ppm F⁻). For children under 7 years of age, supervise use, minimise swallowing and in case of intake of fluoride from other sources, consult a dentist or doctor.

Useful information for diabetics in case of accidental swallowing.
Although bioXtra Dry Mouth products are not intended to be swallowed, the following information may be useful for diabetics in case of accidental ingestion.

Only the following bioXtra products contain Dextrose:
• bioXtra Dry Mouth Oral Gel contains 0,0028 g Dextrose per gram of Gel.
• bioXtra Dry Mouth Mild Toothpaste contains 0,0025 g Dextrose per gram of Toothpaste.

In case of serious incidents, report to the manufacturer and competent authority.

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Sorbitol, Hydrated Silica, Aqua, Glycerin, Sodium Monofluorophosphate, Isoceteth-20, Xylitol, Aroma, Hydroxyethylcellulose, Titanium Dioxide, Cellulose Gum, Glucose, Colostrom Whey, Sodium Saccharin, Glucose Oxidase, Lactoperoxidase, Lactoferrin, Lysozyme, Potassium Thiocyanate, Sodium Benzoate, Limonene, Linalool.

bioXtra Dry Mouth Ultra Mild Mouthrinse/ Mioroladne jemná ústna voda bioXtra pre suché ústa/ Ultra jemná ústná voda bioXtra pre suché ústa/ Apa de gură ultra-delicată pentru gură uscată bioXtra/ Ультрамека вода bioXtra за суха уста/ Супер мягкий ополаскиватель против сухости в рту bioXtra - 250 ml

Aqua, Propylene Glycol, Xylitol, Sodium Monofluorophosphate, Poloxamer 407, Hydroxyethylcellulose, Aroma, Lactoferrin, Lysozyme, Lactoperoxidase, Aloe Barbadensis Leaf Juice, EDTA, Colostrom Whey, Lactic Acid, Sodium Benzoate, Limonene, Linalool, CI42090.

bioXtra Dry Mouth Gel/ Ústny gel bioXtra pre suché ústa/ Ústni gel bioXtra pre suché ústa/ Gelul oral pentru gură uscată bioXtra/ Fen bioXtra за суха уста/ Гель против сухости в рту bioXtra - 40 ml Aqua, Glycerin, Sorbitol, Hydrogenated Starch Hydrolysate, Xylitol, Hydroxyethylcellulose, Glucose, Butylene Glycol, Sodium Polyacrylate, Polyacrylic acid, Colostrom Whey, Glucose Oxidase, Lactoperoxidase, Lactoferrin, Lysozyme, Aloe Barbadensis Leaf Juice, Potassium Thiocyanate, Benzoic Acid.

bioXtra Dry Mouth Gel Mouthspray/ Gelový ústny sprej bioXtra pre suché ústa/ Gelový ústni sprej bioXtra pre suché ústa/ Spray-ul de gură tip gel pentru gură uscată bioXtra/ Спрей с гел bioXtra за суха уста/ Спрей против сухости в рту bioXtra - 50 ml

Aqua, Xylitol, Hydrogenated Starch Hydrolysate, Sorbitol, Hydroxyethylcellulose, Sodium Monofluorophosphate, Sodium Saccharin, Potassium Chloride, Sodium Chloride, Magnesium Chloride, Dipotassium Phosphate, Calcium Fluoride, Colostrom Whey, Lactoperoxidase, Citric Acid, Sodium Benzoate, Sodium Methylparaben, Sodium Propylparaben, Potassium Sorbate.

Lifestream Pharma n.v., s.a.
Rue Jules Bordet, 69
B-7180 Senefels, Belgium
www.bioXtra.info

Distributors/ Distribútori/ Distributori/ Distributori/ Дистрибутори/ Дистрибуторы:
www.bioXtra.info

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