en bioXtra Dry Mouth Gel Mouthspray (50ml)

Please read the following instructions for use carefully, they contain important information.

bioXtra Dry Mouth Gel Mouthspray should only be used in accordance with these instructions or with those given by your pharmacist / health care professional.

If you experience any undesirable effect when using the product, seek medical advice.

Seek advice from your pharmacist / health care professional for any further information or if you are not sure how to use the product correctly.

Do not use this product if the bottle or spray cap is broken or soiled.

WHAT IS BIOXTA DRY MOUTH GEL MOUTHSpray, AND WHEN SHOULD IT BE USED?

bioXtra Dry Mouth Gel Mouthspray is a medical device. The intended purpose is to alleviate the symptoms and effects of dry mouth (xerostomia) whether due to medical conditions, medical treatments or medical interventions.

What is Dry Mouth (xerostomia) and what are the benefits of using bioXtra Dry Mouth Gel Mouthspray?

Natural saliva contains a complex balance of essential components, which protect the sensitive tissues of the mouth and gums.

A reduction or halt in saliva flow can lead to a condition known as dry mouth, clinically known as xerostomia. The effects of dry mouth may include discomfort, pain, difficulty breathing, difficulty with speech and swallowing, and reduction in taste quality.

bioXtra Dry Mouth Gel Mouthspray acts as an effective substitute for natural saliva. It has been formulated to help alleviate dryness in the oral cavity by supplementing the natural moisturising and lubricating systems in saliva with an instant soothing effect. bioXtra Dry Mouth Gel Mouthspray also contains lactoperoxidase, an enzyme found in natural saliva which helps maintain a healthy microenvironment in the mouth.

bioXtra Dry Mouth Gel Mouthspray does not contain alcohol, menthol, or detergents and has no artificial flavourings.

When to use bioXtra Dry Mouth Gel Mouthspray (indications):

There are multiple causes of xerostomia. These include conditions such as:

- Sjögren's Syndrome
- Systemic conditions, such as high blood pressure, diabetes, and auto-immune diseases
- Cancer therapy
- Hormonal imbalance
- Stress and anxiety
- Use of multiple medication

bioXtra Dry Mouth Gel Mouthspray can be used to alleviate the symptoms of xerostomia originating from all known causes.

IMPORTANT INFORMATION & WARNINGS BEFORE USING BIOXTA DRY MOUTH GEL MOUTHSpray

Contains Sodium Monofluorophosphate (150 ppm as F). In case of intake of fluoride from other sources, consult a dentist or doctor.

When not to use bioXtra Dry Mouth Gel Mouthspray (contra-indications):

Do not use bioXtra Dry Mouth Gel Mouthspray if you are allergic to any of the ingredients.

Contains Lactose, not suitable for patients with lactose intolerance.

Risk of allergy:

Contains traces of milk proteins, known to cause allergy in rare cases. In case of a suspected allergic reaction seek immediate medical assistance.

Contains traces of Gluten (<20ppm).

Side effects:

bioXtra Dry Mouth Gel Mouthspray has no known side-effects. However, if you should experience any unexpected side effects related to the use of the product, seek medical advice.

In case of serious incidents in relation to the use of this product, report immediately to the manufacturer and your local competent authority.

Incompatibilities / Interactions with other medication, food or alcohol:

None known

HOW TO USE BIOXTA DRY MOUTH GEL MOUTHSpray

May be used as and when symptoms occur. If you feel an improvement, continue using the product for a few more days before slowly reducing the number of applications. If symptoms return, use for as long as necessary paying attention to the recommended daily dose.

Intended Users:
Adults and Children 12 years and over.
For children under 12 years of age, parent or guardian should seek medical advice before use.

Not suitable for infants. Keep out of reach of children.

Recommended daily dose:
Up to 3.8 ml per 24 hour period.
2-4 pump doses sprayed onto the tongue and into cheeks per application.
Do not exceed 18 pump doses per 24 hours.

Directions for use: bioXtra Dry Mouth Gel Mouthspray is for oral use only.

Spray 2-4 pump doses onto the tongue and into cheeks whenever mouth feels dry. In case of direct contact of nozzle with mouth surfaces, wash after use. Repeat when dry mouth symptoms return, without exceeding the recommended daily amount. Close cap after use.

When used in a clinical environment, follow good hygiene practices. Properly place the patient in an upright or semi-upright position before applying the spray.

Avoid drinking/brushing teeth just after application.

Period After Opening (PAO) and Expiry Date:

May be used safely for up to 3 months after opening. The expiry date can be found on the bottle and the box and is expressed as year-month-day (YYMMDD). It overrides the period after opening.

What to do in case of accidental ingestion of large quantities:

bioXtra Dry Mouth Gel Mouthspray is safe when used as instructed. It contains sorbitol (5%) and is not intended to be ingested in large quantities. Doing so may have a laxative effect. In case of accidental ingestion of large quantities, or accidental ingestion by children under 12, seek medical advice.

Do not use this product if the bottle or spray cap is broken or soiled.

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